

AED MAINTENANCE CHECKLIST

- Each month go to the AED, open the cabinet and take the AED out – close the cabinet to stop the siren.
- Check that the “status indicator” on the front of the AED (green light) flashes green every 5 seconds or so.
- If the status indicator flashes red, something needs to be addressed with the AED – contact one of the recommended supply partners for Stryker products.
- Press the “On” button to turn the AED on and make sure it starts speaking – if it starts speaking then turn it off. If nothing is heard, contact one of the recommended supply partners for Stryker products.
- Open up the carry bag, turn the AED over and check the AED PadPak expiry date -written on the PadPak itself. This should be a date in the future. If it is not, you need to get a new PadPak for the AED.
- While the AED is out of its carry bag, do a visual inspection for any cracks or excessive dust/dirt. If there are cracks, please report to one of the supply partners for Stryker products. If there is dust/dirt, please use a dry or slightly damp rag to clear the dust/dirt.
- Replace the AED in its carry bag and return to the cabinet.

We would advocate that users make a note of the date of the inspection, and perhaps even take a photo of the PadPak expiry date on their phone (which will be date-stamped), and/or take a video of the green status light flashing.